VERMEIREN

Liftchair Ontario

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES

Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: H, 2016-08

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© Vermeiren GROUP 2016

EN Ontario V

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the lift chair, read the instruction manual carefully: it will familiarize you with this product.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for instructions for use.

1. Technical Specifications

Ontario V		
Description	Dimensions	
	1 motor	2 motors
Total width	800 mm	845 mm
Dismantled width	Not applicable	
Dismantled width (backrest)	700 mm	670 mm
Seat width	465 mm	480 mm
Height armrests	150 mm	140 mm
Seat height (Sit position)	500 mm	500 mm
Seat height (Lift position)	550 mm	550 mm
Total height (Sit position)	1160 mm	1145 mm
Total height (Recline position)	1000 mm	645 mm
Total height (Lift position)	1475 mm	1500 mm
Dismantled height (without backrest)	630 mm	645 mm
Dismantled height (backrest)	810 mm	830 mm
Total length (Sit position)	840 mm	960 mm
Total length (Recline position)	1520 mm	1880 mm
Total length (Lift position)	950 mm	885 mm
Dismantled length (without backrest)	810 mm	810 mm
Dismantled depth (backrest)	320 mm	300 mm
Seat depth	450 mm	530 mm
Height backrest	695 mm	670 mm
Angle backrest (Sit / Recline / Lift)	4° / 43° / -26°	10° / 75° / -16°
Angle legrest (Sit / Recline / Lift)	-90° / +12° / -38°	-90° / +2° / -27°
Angle seat (Sit / Recline / Lift)	+9° / +18° / -29°	+9° / +9° / -18°
Height adjustment (measured on back side of liftchair)	670 mm	500 mm
Distance from wall	450 mm	450 mm
Total weight (N.W.)	49.64 kg	53.34 kg
Mass of heaviest part (that can be dismantled or removed) (N.W.)	42.66 kg	47.00 kg
Masses of parts that can be dismantled or removed. (N.W.)	Backrest: 6.98 kg	Backrest: 6.34 kg
Weight user	150 kg	
Noise level	<53 Db	
Power	220/230 V	
AC/DC Adapter	Input: AC 100-240 V, 50/60 Hz Output: DC29V Max. 10%, max 2 min	
Degree of protection actuator	IP42	

Subject to technical alterations – vH - 2016-08

Power rating, Duty cycle actuator	DC24V, Max. 10%, max 2 min/18 min
Degree of protection manual control	IP42
Storage and use temperature	+5 °C to +41 °C
Operating temperature of the electronics	-10°C to +40°C
Storage and use humidity	30% to 70%

Deviations in sizes and weight (15 mm en 1,5 kg) are possible.

2. Explanations of symbols

Warning: Do not Pinch hands / fingers

CE conformity

Maximum mass

Indoor use

Separate recovery and recycling of electric and electronic devices

Protection class II

Type designation

3. Components

- 1 = Backrest
- 2 = Armrests
- 3 = Seat
- 4 = Legrest
- 5 = Manual control
- 6 = Supply cable
- 7 = Headrest cover
- 8 = Leg Levelers

4. Location identification plate

Place the liftchair with the manual control in the lift function, the lower frame will be visible. On these lower frame you can find the identification plate with CE logo and serial number.

5. Accessories

Standard is the liftchair Ontario single motor delivered with two wheels for mounting on the back side of the lower frame and two small leg levelers for mounting on the front side of the lower frame. Ontario Dual motor has four wheels. Option - two big leg levelers for back side of the lower frame (Both for single or dual motor).

Use only Vermeiren approved accessories.

6. Entrapment zones

In the figure below the different entrapment zones (area 1 until 4) of the liftchair are indicated. **Make sure that your fingers, jewelry or clothes cannot become pinched in these zones.**

7. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- Seat, armrests, legrest and lower frame
- Backrest
- Headrest cover
- Single motor - 2 wheels + 2 leg levelers (Option: 4 leg levelers)
- Dual motor - 4 wheels (Option: 2 wheels and 2 leg levelers)
- Manual control
- Manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

8. Assembly


The liftchair is delivered fully assembled with exception of the backrest and the wheels, which are easy to mount on the liftchair.

When transporting the liftchair in a vehicle you can dismount the wheels and backrest or place the complete liftchair in the vehicle.

Store the separate parts (wheels and backrest) safely.

Grasp the seat on the lower frame (not on the armrests, backrest) and place these in the vehicle with 2 persons.

8.1. Mounting backrest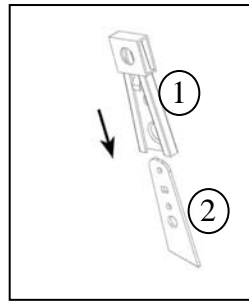

Place the backrest with the two sleeves ① on the plates ② mounted on the seat.

Push on the topside of the backrest until it click in place.

Check that the backrest is firmly fixated, by pulling softly on the backrest. If the backrest come loose repeat the steps above.


Mount the loose upholstery of the backrest with the velcro strips to the lower frame.

8.2. Removing backrest

Remove the loose upholstery of the backrest with the velcro strips from the lower frame.

Use a flat head screwdriver, and place these against the locking clip. Pull these locking clip upwards. At the same time pull on the backrest until the two connections plate come loose from each other.

Lift the backrest out the lower frame.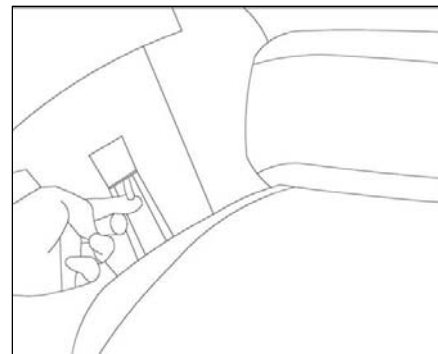

8.3 Mounting wheels

Standard are two wheels delivered with your liftchair. Mount these two wheels on the back side of the lower frame.

9. Use

These liftchair can be used for elderly people and persons with weak legs. The liftchair gives a better seating comfort and it is easier to come out the chair.

The liftchair is only for indoor use.


The liftchair will touch the ground when the liftchair is in the seating position (liftchair can be moved).

The wheels will be blocked when using the lift-function (liftchair cannot be moved).

You can place the liftchair in a lift function or lay back function with the buttons on the manual control. These shall be only operated by the intended user.


10. Transportation

Do not pull on the backrest or armrests cushions to move your liftchair, even over a tiny distance (100 or 150 mm) to another place. These cause tear to the stitching.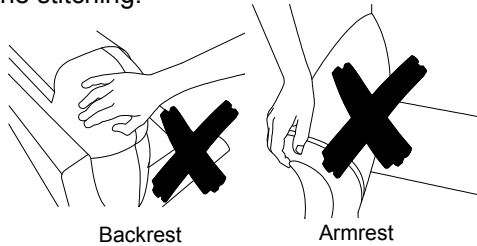

Backrest

Armrest

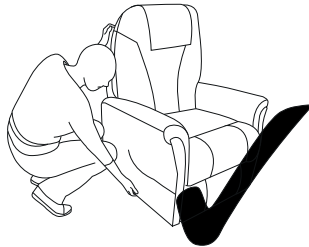Use the lower frame of the liftchair (side and bottom of the liftchair)

Do not pull on the armrests to lift yourself out the chair.

To lift yourself out the liftchair, place the chair in the lift-position with the handset.

11. Use of the manual control

1 Motor

LIFT FUNCTIONS
Stepless adjustment from sit-position to lift-position

LAY BACK FUNCTION
Stepless adjustment to stretched position

2 Motors

LIFT FUNCTIONS
Stepless adjustment from sit-position to lift-position

LEGREST INCLINATION
Stepless adjustment from legrest

SITTING POSITION
Stepless adjustment back to sit-position

LAY BACK FUNCTION
Stepless adjustment to stretched position

→ When the liftchair is not being used, leave it in the sitting position and not in the elevated position.

12. Safety rules

- ⚠ **Please do not exceed the maximum users weight of 150 kg.**
- ⚠ **Do not allow children to be under the chair, use the chair or play with it because it might cause injuries when the legrest are folding.**
- ⚠ **If the liftchair blocks in the lying position, you must ask assistance to leave the chair.**
- ⚠ **It is strongly recommended not to operate the liftchair near a desk, a wall or other objects too close to it, as not to cause damage by tilting the backrest or the legrest.**
- ⚠ **For prolonged use of the liftchair motor (about 2 min), you'll have to wait a few minutes before reusing it. This is needed to avoid overheating the liftchair motor.**
- ⚠ **Do not move the liftchair:**
 - when the liftchair is in the lying position;
 - when the liftchair is connected to the power and/or in service.
- ⚠ **Be careful not to sit on the backrest, armrests or on the legrest.**
- ⚠ **Ensure that no cables become squeezed.**
- ⚠ **Risk of injury - Surfaces can assume the environment temperatures.**

13. Electromagnetic compatibility EMC

Your liftchair has been tested according EN 61000-6-3: 2007, EN 61000-6-1: 2007 for EMC-compliance.

To reduce the effect of electromagnetic sources of interference, please read the following warnings:

- ⚠ **The liftchair might disturb the operation of devices in its environment that emit electromagnetic fields.**
- ⚠ **Should any unintended movements occur, remove the power plug from the liftchair from the wall socket.**

Disruption of electrical equipment from electromagnetic fields in the surroundings can not be entirely avoided.

In the presence of very strong or long-lasting disruptive fields, the electrical systems might break down completely and suffer permanent damage.

14. Maintenance, Inspection, cleaning

For the maintenance manual of the liftchairs refer to the Vermeiren website: www.vermeiren.be.

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles and other products: 2 years (batteries 6 months) and multipoosition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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Vermeiren Suisse S.A.